

END-OF-YEAR EQUIPMENT SHUTDOWN PROTOCOL



STERILISATION ROOM

- DRAIN & CLEAN** The internal water reservoirs on your autoclave.
- TURN OFF** The water supply to your Demineraliser/Reverse Osmosis (RO) Unit.
- TURN OFF** The autoclave & disconnect the power plug from power point.



DENTAL CHAIR

- REMOVE** The couplings & motors to prevent existing debris in the lines causing blockages.
- FLUSH** All water lines with a full bottle of water with an ICX Tablet. Once the bottle is empty continue to flush all the lines until only air is coming out. Don't forget the triple syringes! This will minimise biofilm growing.
- ASPIRATE** The recommended suction cleaner through your vacuum lines & then leave all cannulas open & aspirating air for at least another 5 minutes to dry the lines.



PLANT ROOM

- CLEAN** The suction filter.
- DRAIN** The compressor reservoir (tank), there is typically a valve on the bottom of the tank. Also drain the condensate bottle (if fitted).
- TURN OFF** The compressor, you don't want the compressor running all holidays.



COMMON

- TURN OFF** Turn off & unplug ALL equipment.